

ESSENTIALLY BETTER



Elvido

The versatile care bed range



International Edition

STIEGELMEYERGROUP



The Elvido family: Tailor-made beds

High quality of life and good patient care without the need for liberty-restricting measures are important goals in the care sector. Great achievements have already been made here: in 2017, the Medizinischer Dienst des Spitzenverbandes Bund der Krankenkassen (medical services of the health insurance umbrella organisations) discovered that the use of liberty-restricting measures had significantly declined from 12.5 to 8.9% since 2013. Many establishments want to reduce this quota even further, to below 5%.

The comfortable beds from the Elvido range can help you in this endeavour. Tailor-made beds for many types of application can be created from the range of available models and flexible equipment options. Split safety sides or safety sides at the head end only, low-height beds for fall protection or the Out-of-Bed system offer an ideal balance between protection and mobility. The elegant, homelike style of the Elvido range ensures the residents' well-being and helps them feel at home.

Safe working
load at least
225 kg

Low-height beds



Elvido vervo

- **Height adjustment:**
approx. 25 – 80 cm
- **Fall prevention:**
 - Low-height bed for protection without liberty-depriving measures
 - OpenBus technology for installing the Out-of-Bed system
 - Follows the Werdenfels approach



Elvido movo

- **Height adjustment:**
approx. 28 – 83 cm
- **Manoeuvrability:**
 - Large 100 mm double castors
 - Easy to manoeuvre even in cramped areas and when transporting over longer distances
 - Effortlessly overcomes smaller obstacles such as door thresholds

Contents

4	Model variants	19	24-volt-System
8	Decor in the chassis	20	Bed extension
9	Mobilisation und Ergonomie	22	Operating concept
10	Vario Safe	24	e-help
12	Safety sides	26	Accessories
16	Mobilisation and ergonomics	28	Technical data/Features
18	Mattress bases	30	The Stiegmeyer-Group



Mechanical bed



Elvido rega

- **Height adjustment:**
approx. 39 – 80 cm
- **Suitable for psychiatric units wards:**
 - Mechanical bed
 - Excellent stability
 - Easy adjustment by operating hydraulic foot pump

Heavy-duty bed



Elvido forto

- **Height adjustment:**
approx. 39 – 80 cm
- **Safe working load:**
260 kg for heavy residents
- **Mattress base width:**
100 cm
- **Castors:**
125 mm castors, 32 mm wide



Elvido vervo



The versatile low-height bed to meet exacting requirements

The Elvido vervo is the model with the largest height adjustment range and is therefore also suitable for particularly intensive care situations. It offers the best possible protection from fall injuries and an optimal working height, even for tall care staff. Thanks to numerous optional features, this model can become a personalised, tailor-made bed.

- Height adjustment range: approx. 25 – 80 cm
- All residents, no matter how tall or short, have secure floor contact when leaving the bed; orientation and balance are heightened thanks to tactile perception by the soles of the feet
- OpenBus drive system, a standard feature, allows the use of the optional Out-of-Bed system

Low-height bed

External dimensions/height adjustment range:

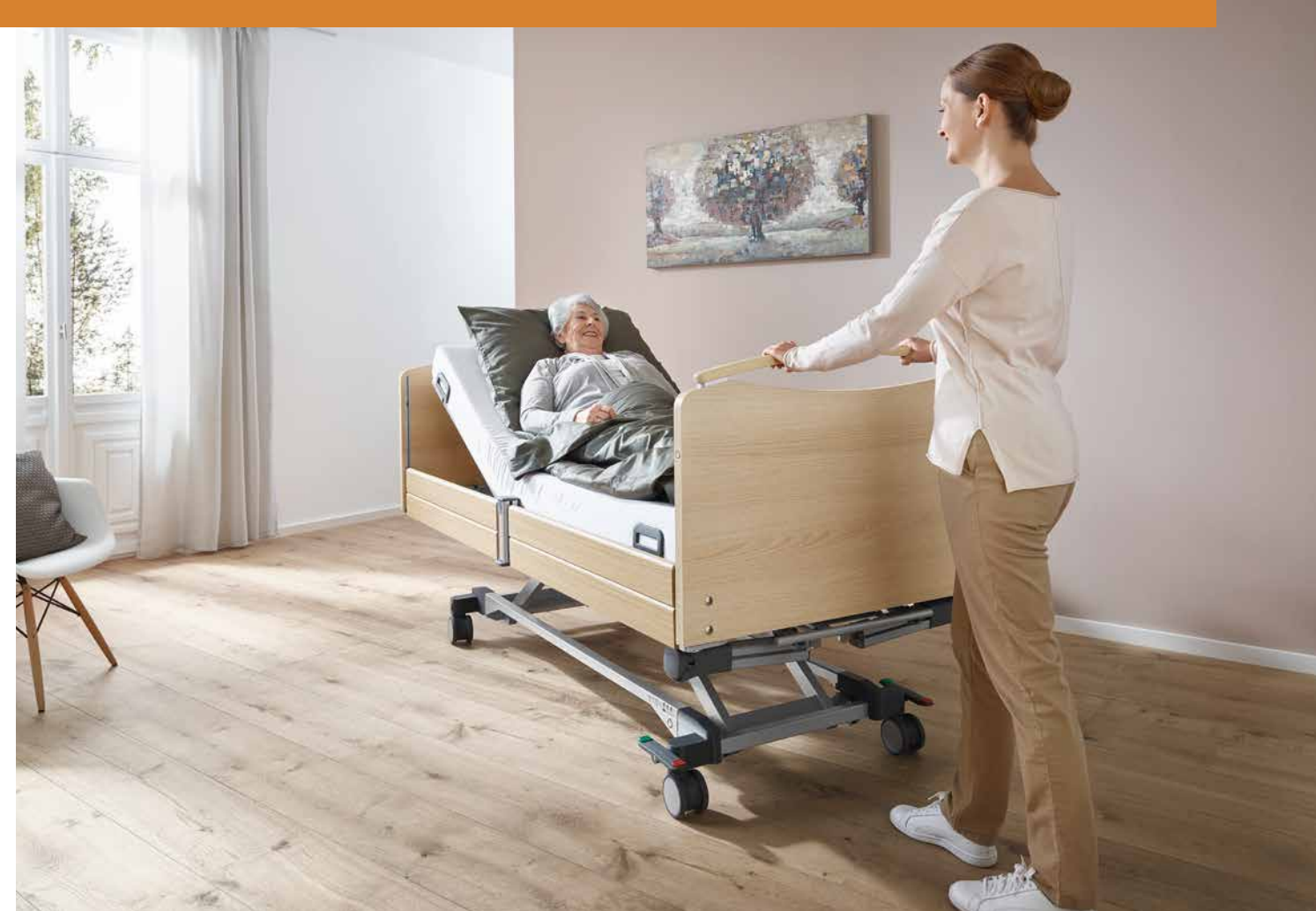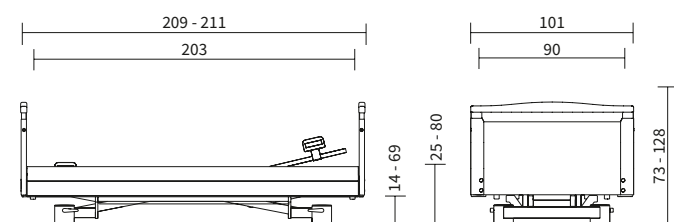

Elvido movo


The particularly easy to move low-height bed

The Elvido movo is particularly easy to move and manoeuvre in confined spaces. This makes it the ideal choice for care and rehabilitation centres with exacting logistical demands regarding bed preparation and for residents who are unable to leave the bed in daily life.

- Large 100 mm double castors for easy manoeuvring
- The bed can be effortlessly moved to another care room or taken to a hygiene area for reprocessing
- In the bed, immobile persons in need of care can take part in daily social life in the common room or on the patio
- Directional castor for even easier manoeuvring

Low-height bed

External dimensions/height adjustment range:

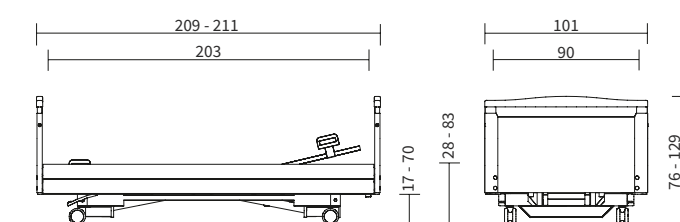




Elvido rega

The ideal care bed for psychiatric facilities

The mechanical Elvido rega care bed offers comfort without cables and uses practically no components that can be removed without tools. This makes a significant contribution to the safety of residents who have serious psychiatric conditions or dementia.

- Height adjustment from 39 bis 80 cm using an optional removable pump pedal
- Robust metal mattress base with welded-in mesh base
- Optional securing brackets on the rests and frame profiles of the mattress base
- Ideal for locations with no power supply

External dimensions/height adjustment range:

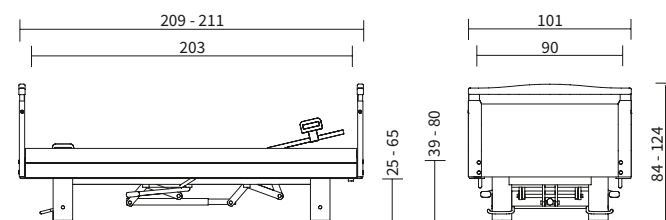

Mechanical
bed




Elvido forto

More comfort for tall and heavy residents

The problem of overweight and obesity is increasing globally. At the same time, the average height of people in Germany increased by 15 cm in the course of the 20th century. Tall and heavy people also present a challenge in the care sector. The Elvido forto can help here.

- 260 kg safe working load for heavy residents
- Mattress base width of 100 cm and easy-to-handle bed extension for more space and comfort
- Ergonomically beneficial working height of up to 80 cm makes care work easier
- Large 125 mm castors allow easy movement of the bed, even when heavily loaded

External dimensions/height adjustment range:

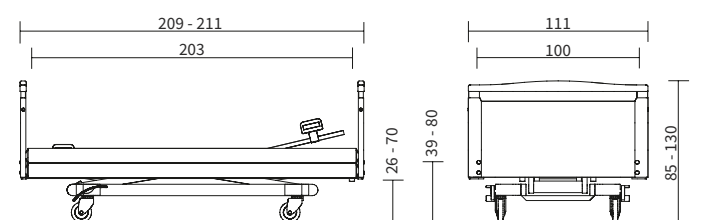



The optional cover matches the decor colour of the bed.

Chassis

with harmonious lines

The customisable wooden surround and the chassis give the Elvido range beds a cosy feel. The ends of its longitudinal struts are fitted with **four plastic covers** as standard. With their silver-grey colour, they exactly match the tone of the chassis. The covers encase mechanical parts and make the lines of the bed more harmonious. Additionally, the smoother surface can be cleaned even more easily and thoroughly. If required, the plastic elements can be removed without the use of tools.


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
Upon request, more **complete covering** is available. The two plastic elements cover the entire front faces of the chassis and can be supplied in many homelike wood decor colours. Those who only need a covering for the headboard or the footboard can also just order a single element. The decor cover conceals the technical parts of the care bed, giving it a particularly homelike appearance. It is also easy to clean and remove.





▲ Four plastic covers conceal the ends of the longitudinal struts as standard.

ESSENTIALLY BETTER

Attractive colours to choose from

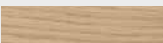-  Lindberg Oak
-  Style Oak Cinnamon
-  Havana Cherry
-  Natural Beech
-  Chalk
-  Silver grey

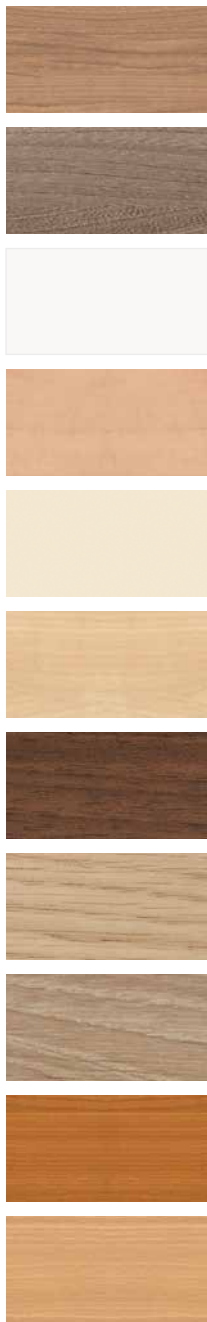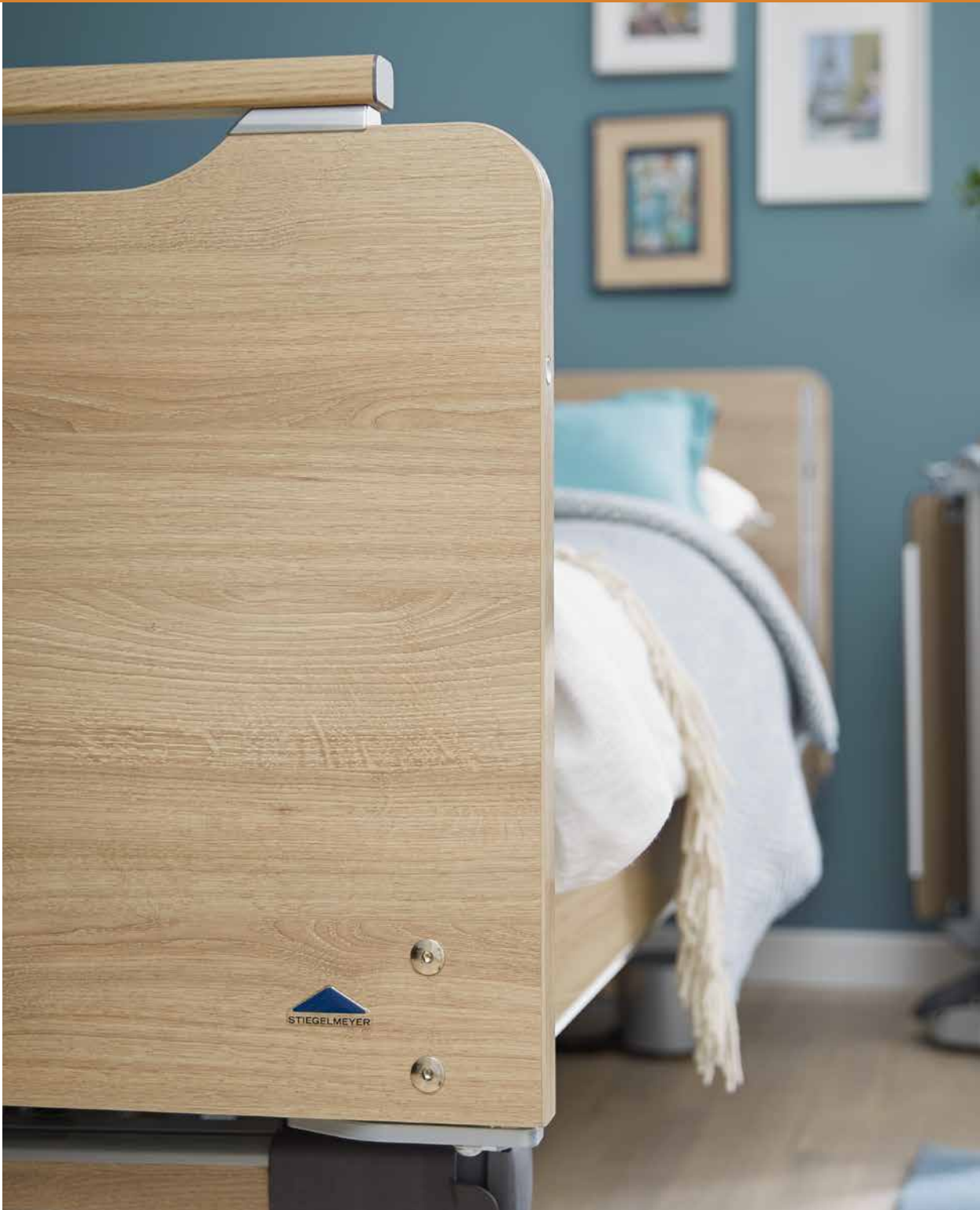

Headboards, footboards and decors

A feeling of home whatever your interior design

Straight, curved, or with a handle bar – attractive forms and shapes are available for the headboards and footboards of the Elvido beds. And the various homelike wood decors adapt every bed ideally to suit its surroundings. Residents will soon feel at home in their beds.


Further information can be found in our brochure entitled “Head and footboards and decors”



Elvido with Vario Safe – the best solution for standard-compliant safety

Choose the Elvido with the split safety side or without a safety side and benefit from our flexible Vario Safe system. The Vario Safe principle is as simple as it is impressive: The safety side elements, headboards, footboards and side panels can be attached and removed without tools. This is possible thanks to easy-to-use fasteners that snap into place securely and cannot be accidentally

operated by the resident. Headboards and footboards can be delivered on request with attractive fabric or synthetic leather upholstery. Beds with Vario Safe can be adapted to the changing needs of residents in just a few minutes. On request, nursing homes can order their beds in a basic version and add further parts at a later date.

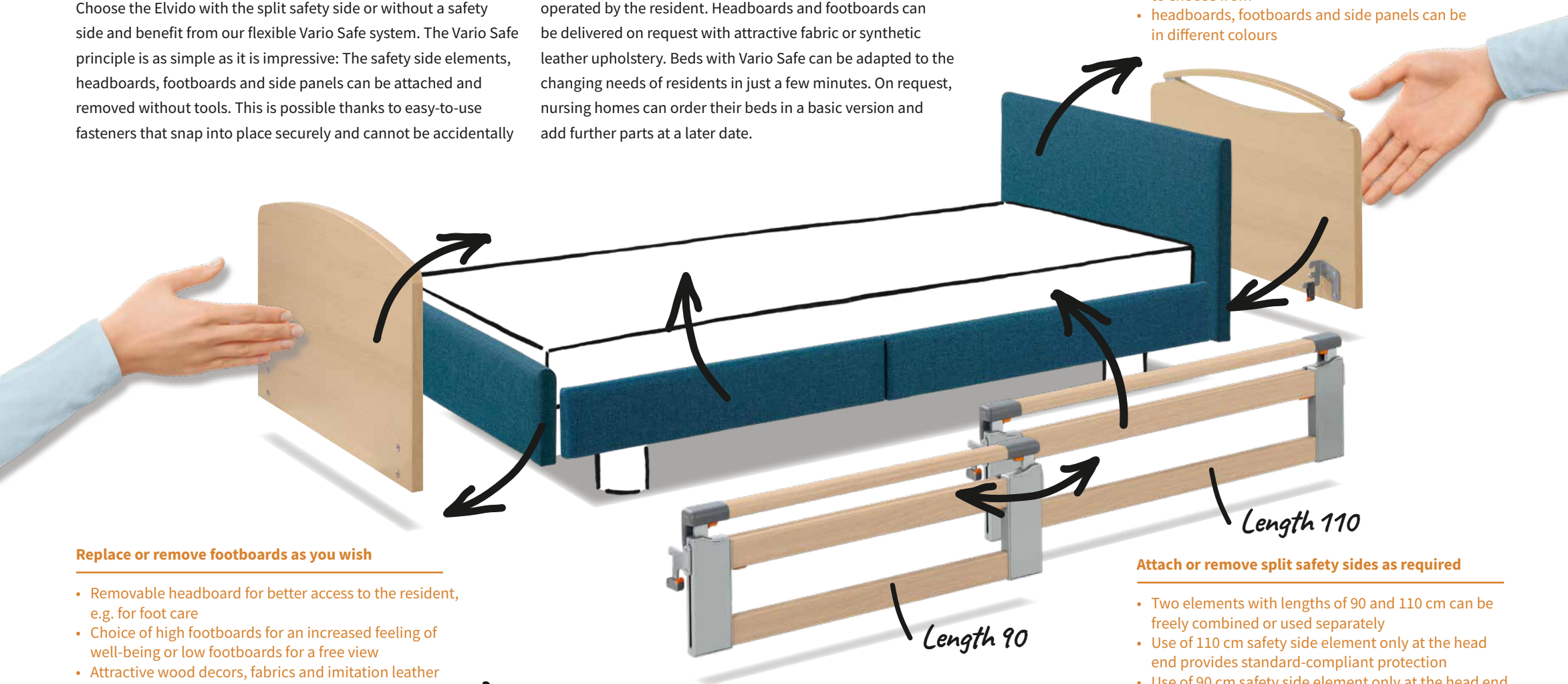

Replace or remove footboards as you wish

- Removable headboard for better access to the resident, e.g. for foot care
- Choice of high footboards for an increased feeling of well-being or low footboards for a free view
- Attractive wood decors, fabrics and imitation leather options available

! Safety sides can be freely combined: 110/90 or 90/110

Headboards with Vario Safe



Modular design and adaptable without tools

Exchange headboards in seconds according to requirements

- Different homelike designs, with or without handles
- Attractive wood decors, fabrics and synthetic leather to choose from
- headboards, footboards and side panels can be in different colours

ESSENTIALLY BETTER

With Vario Safe more flexibility and safety

Beds with Vario Safe meet the different protection needs of the residents and can be customised to suit any taste. The flexible safety side provides standard-compliant protection without restricting the patient's mobility.



◀ All Vario Safe elements are inserted into the bed frame and secured with easy to use locking devices.




◀ To release the fastening again, pull the orange safety catch toward you and press the locking device down.

Attach or remove split safety sides as required

- Two elements with lengths of 90 and 110 cm can be freely combined or used separately
- Use of 110 cm safety side element only at the head end provides standard-compliant protection
- Use of 90 cm safety side element only at the head end gives a feeling of security and orientation and leaves plenty of room for getting in and out of bed

Decors with Vario Safe

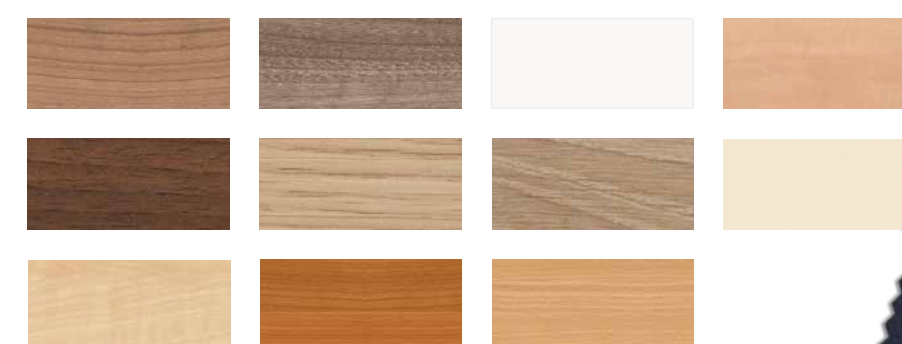

Please contact us for more information about other fabrics and imitation leather.

Please observe the manufacturer's instructions when cleaning the fabrics and synthetic leather





Safety sides

The ideal balance between protection and freedom

The Werdenfels approach is also reflected in the safety sides of the Elvido range. Two types of split safety sides are available for selected model variants. They ensure that residents can enjoy an optimum balance of protection and freedom.

The bars of the **combinable safety side** are attached to a central support which also serves as a mobilisation aid and provides the resident with a secure hold when standing up or sitting down. When no longer required, the support can be easily removed and stored in a holder under the bed. On request, the combinable safety side can be configured at the head end only - together with the centre support it still covers more than 50% of the mattress base and meets the requirements of the relevant standards. As its name suggests, it can be combined with a continuous safety side on the other side of the bed.


The telescopic elements of the **split safety side** can easily be raised in two stages. In the first stage, the resident is provided with an orientation aid, in the second stage, complete protection. The 110 cm long elements at the head end of this model also satisfy the requirements of standards when used by themselves. Homelike panels can be attached to the foot end if desired. The telescopic split safety side is attached using the Vario Safe system.

The standard **full-length safety side** offers maximum protection and, at a height of 41 cm, is highly suitable for using with an anti-decubitus mattress. For independent users, such as those in rehabilitation facilities, the **EasyGrip** model is a good alternative – slender rails at the end give better orientation. All Elvido beds are available without safety sides as well.



- ◀ The safety side at the head end always leaves enough room for getting in and out of bed unhindered.
- ▼ The centre support of the split safety sides also serves as a practical mobilisation aid.

- ▲ The full-length safety sides supplied as standard offer maximum protection against falls.
- ▼ The safety side can be set diagonally as an orientation aid for the resident without restricting his or her mobility or view of the room.

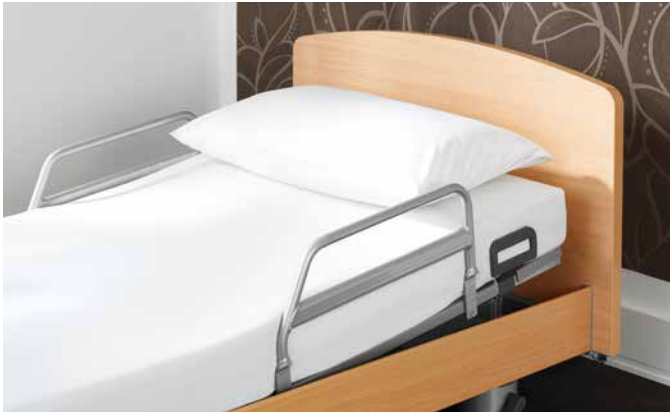

- ▲ EasyGrip is a rail that gives the resident a better feel for the dimensions of the bed and also provides support when leaving the bed.

- ▲ The telescopic split safety side flexibly adjusts to suit all needs.



Split safety sides

Individual safety in every function

The Vario Safe system is available with the Libra care bed and with the Elvido for the split safety side. This enables very **precise settings** to be made that are closely tailored to the changing needs of the resident.

The individual elements of the split safety side can be **raised in two stages**. In the first stage, they provide the occupant with support and orientation, and in the second stage, they provide reliable protection. A diagonal position also ensures a good feeling of safety. If the patient is able to leave the bed whenever they like, we recommend only **raising the safety sides at the head end**. In this case, the foot-end elements can also be removed and replaced by homelike panels. The streamlined wood-metal appearance of the safety side ensures excellent hygiene properties.

The split safety side can be raised and locked securely into place in **one easy movement**. The elements are lowered again by pressing the release buttons under the bars.

ESSENTIALLY BETTER

Standard-compliant protection

- In a mattress base measuring 200 cm, the **split safety side** on each side consists of a 110 cm and a 90cm long element.
- These elements can be used **interchangeably** at the head and foot ends.
- If the 110 cm element is attached at the head end, it covers **more than 50% of the mattress base** and is therefore also a standard-compliant safety side on its own.
- The 90 cm element does not offer standard-compliant protection, but it provides **support and plenty of room** to get in and out of bed.
- If the **bed extension** is used, two 110 cm elements can be attached.



▲ If an object is in the way when the bed is lowered, the split safety side gives way flexibly.




▲ Simply press the button to lower the safety side.



! Safety sides can be freely combined: 110/90 or 90/110



▲ The 110 cm element also provides standard-compliant protection when used on its own at the head end.

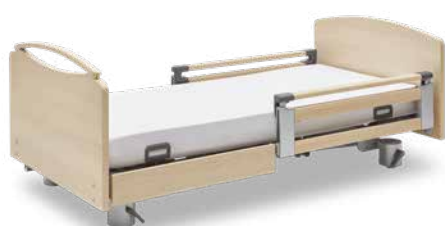

▲ The 90 cm element provides support. All elements can be raised in two stages.



▲ Raising the safety side diagonally gives the resident a feeling of safety and orientation.

ESSENTIALLY BETTER

Vario Safe - flexibility for everyday care

Everybody is talking about adaptable care beds – our Vario-Safe system provides you with the best and most sophisticated solution. Here is a list of all the advantages:


Headboards and footboards

Combine a wide range of attractive designs, wood decors and fabrics - attachable in seconds.



Panels

Cheerful panels in a wood or upholstered look can be fitted instead of the individual safety side elements.

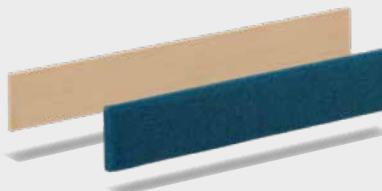

Safety sides

The split safety side enables standard-compliant protection without restricting the resident's mobility.





◀ The Elvido vervo bed has a particularly low setting that protects against fall injuries without the need for liberty-restricting measures.

Low-height beds for fall prevention

Mobilisation and ergonomics

Always at the right height for ease of movement

Getting up and getting into bed are particularly problematic moments in the day of a person in need of care. On the one hand, we want the resident to make these movements independently for as long as this is possible. On the other hand, these are precisely the situations that involve a high risk of falling. Cognitive limitations, reduced muscle mass, a weaker sense of balance or side effects of medicines can unsettle older people.

With their large, **stepless height adjustment range**, the Elvido range beds assist residents and carers alike. Falls can never be prevented with absolute certainty, but, particularly with the low-height Elvido vervo bed, the risk of injury at the lowest position is significantly reduced.

This often means that liberty-restricting measures such as restraint systems or full-length safety sides can be dispensed with as a result.

The optimal ergonomic **height for getting in and out of bed** can be adjusted as required using the handset (or the pump pedal in the case of the Elvido rega). For many people, this is below 40 cm – the feet then make good contact with the floor and the proprioceptors are stimulated. At the **maximum height** of approximately 80 cm, care staff can work with a straight back and thus protect their health.

◀ The large height-adjustment range allows care staff to work without putting strain on their backs.



◀ The hygiene mattress base is easy to remove and thoroughly clean by hand.

▼ If the use of patient restraints is unavoidable, the cut-outs in the hygiene mattress base can be used for securing belts.



Mattress bases

The basis for healthy and restful sleep



▲ Hygiene mattress base

The Elvido **hygiene mattress base**, supplied as standard, makes thorough bed reprocessing particularly easy. Deep venting slits in the polypropylene promote a healthy microclimate in the bed and beneficial sleep. All types of special mattress can usually be used without an underlay mattress, so that the safety sides are still high enough to ensure proper protection.



▲ Comfort mattress base – flexibly supported base components with pivot springs

There is also a **comfort mattress base** available, which has even better characteristics. This comprises 30 flexibly supported base components with 80 pivot springs of varying stiffness. The comfort mattress base adjusts exactly to the contours of the body and improves the pressure-reducing effect of the mattress. The comfort mattress base is slightly higher than traditional models, so you will need to check whether the protection height of the safety sides is sufficient in combination with your mattress.



▲ In the 24-volt system, a transformer plug supplies a safety extra-low voltage at the socket.

Energy-saving and safe

Modern drive systems

A high level of safety, significantly reduced electricity costs and effortless operation: The optional 24-volt system is an attractive choice for the Elvido bed.

A transformer plug supplies a 24-volt **safety extra-low voltage** directly at the power socket. There are therefore no components carrying 230 volts anywhere in the vicinity of the resident, which means that he or she enjoys a higher level of safety.

Power consumption drops by up to 93%. There is a floor-mounted power pack available to use the 24-volt system with international mains plugs.

The Elvido is equipped with a reliable **230-volt drive system** as standard. With matching selectable mains plugs, it is therefore ready for international use wherever it is needed.



For the 230-volt system, 6 types of mains plugs are available, which comply with various international standards.

ESSENTIALLY BETTER


A perfect fit everywhere




The 230-volt drive system is available with suitable mains plugs for many international markets.

ESSENTIALLY BETTER

Strong advantages

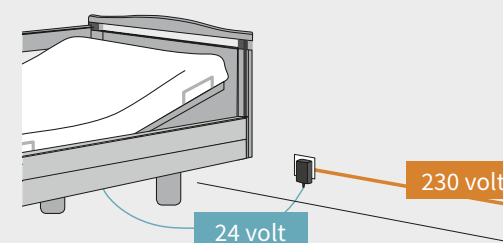

- Safety extra-low voltage starting from the transformer at the power socket
- No 230-volt components on the bed
- The optional rechargeable battery makes the care bed independent of the mains power supply in an emergency

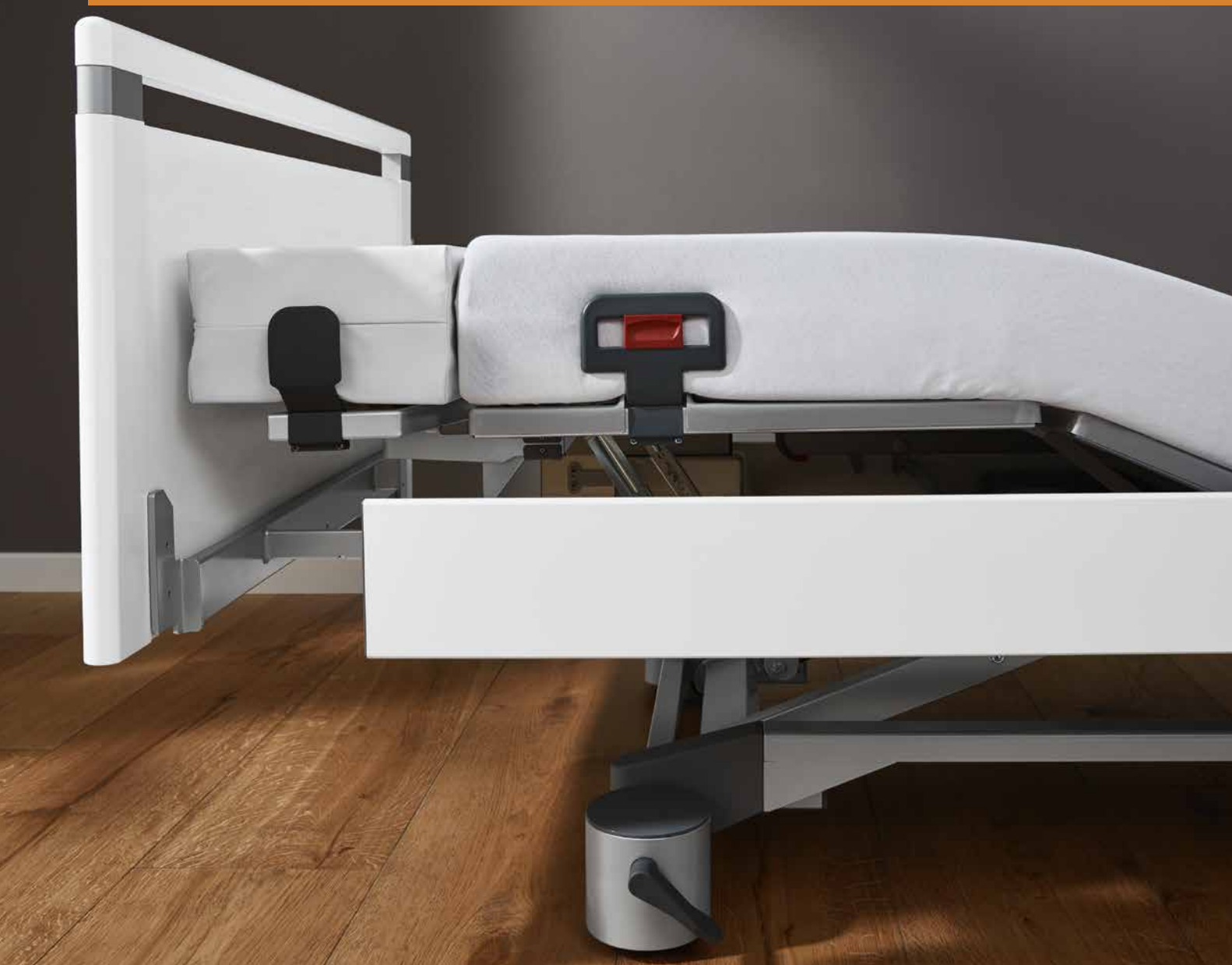

- ▲ If the bed does not have safety sides along the entire length, then the integrated bed extension can be used without any further accessories. The gap in the side panels at the foot end is standard-compliant. It can be closed with filler pieces, if desired.



- ▲ With the telescopic bed extension, the bars and side panels can be simply pulled out to extend them.



- ▲ Longer safety side bars and filler pieces for the side panels are available for the integrated bed extension, if required.

Bed extension

More room and comfort for residents in just a few easy steps

The user-friendly bed extension for the Elvido range improves the comfort of tall residents. All of the models are equipped with an integrated 20 cm **extension of the bed frame as standard** (exception: Elvido rega). If the bed has no safety sides or has safety sides only at the head end, this extension can be used, whenever required, without any further fittings. If safety sides are needed along the entire length of the bed, longer bars can be supplied.

The optional **telescopic bed extension** is even more convenient. This allows the bed frame, safety side bars and side panels to be easily extended without using any tools.

We offer two options for extending the mattress base. The first is a **mattress piece** that always remains securely fixed in its position at the foot end. The second is an **extension of the lower leg rest**, which moves with the leg rest when adjustments are made. This allows even tall people to use the bed in a health-promoting position with the legs raised.



- ▲ A mattress piece, which remains fixed in position, can be inserted when the mattress base is extended.



- ▲ The optional extension for the lower leg rest moves together with the leg rest when moves together are made.

- ▲ The lower leg rest extension can be hooked into place, without needing to use tools. A corresponding mattress piece is available.



ESSENTIALLY BETTER

EasyCare relieves the strain on care staff

- The LCD handset contains **three control levels**: for residents, care staff and technicians.
- Care staff can **block certain adjustment functions for the residents**.
- With the **EasyCare** function, these functions can be used at the care staff level without changing the blocking status for the resident. This saves care staff a lot of time and increases the level of safety.



The LCD handset has just three buttons and an illuminated display, giving all users optimum control of the bed.



▲ The standard handset uses intuitive pictograms to indicate the different adjustment options.

Innovative operating concept

Handset for customised requirements

The Venta is operated using one of a selection of handsets. The most convenient version is the optional **LCD handset** with only 3 buttons, EasyCare and a backlit display. It has 3 control levels: for residents, care staff and technicians. The levels can be unlocked and set with the help of a magnet on the bed.

The **standard handset** displays all the adjustment positions with simple pictograms. The button rows can be selectively locked. A separate **Trendelenburg handset** is available for quickly moving the bed into the Trendelenburg position; this ensures that the resident cannot mistake this function for the reverse-Trendelenburg position.



▲ Trendelenburg handset




The under bed light helps with orientation in critical situations, such as on the way to and from the bathroom at night.

e-help

Digital assistance systems

Digital assistance systems are a modern solution for reducing liberty-depriving measures, protecting residents and at the same time supporting care staff, especially during night shifts. Under the name “e-help”, Stiegemeyer is constantly developing these systems further.

The focus here is on the **Out-of-Bed system**. A load sensor in the backrest registers when a resident gets out of bed. The system also has an optional feature that immediately switches on an under bed light or the reading lamp for better orientation in the dark. After a specified time interval, it notifies the care staff via the existing in-house call system.

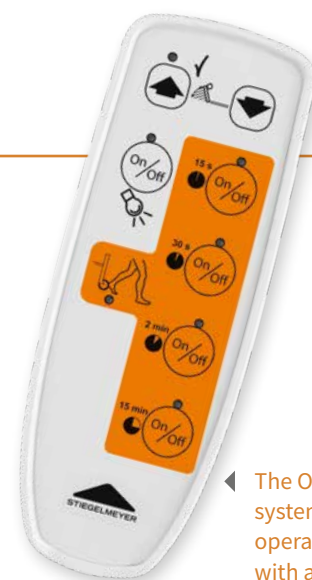

The Out-of-Bed system can be operated very easily with a handset.



If the resident leaves the bed, the system notifies the care staff via a signal light in the corridor, for example, or an acoustic alarm in the duty room.

ESSENTIALLY BETTER

The Out-of-Bed system

- Orientation with the optional under bed light
- Significantly reduced risk of falls at night without the need for restraints
- Care staff are notified after a set time of between 15 seconds and 15 minutes, which avoids frequent false alarms immediately after someone gets out of bed
- Immediate support for residents who are susceptible to falls



The resident leaves his or her bed and, in doing so, triggers the visual signalling process

Visual signal in the care room

Visual signal in the corridor

A visual and acoustic signal is activated in the care room



▲ The elements of the split safety sides can be covered separately.



Additional products can be found in our accessories brochure

◀ The Stella reading lamp is an elegant design alternative.

▼ The Sola radiates a pleasant light and generates very little heat.



Accessories

Tailoring the Elvido to your taste

All Elvido models can be transformed into tailor-made beds using original accessories for the highest level of comfort and to meet very specific requirements. The central focus is on improving safety for the resident, ensuring a higher quality of everyday life and protecting the beds from damage.



▲ All-round protection against impact injuries can be provided for residents by fitting protective covers on the headboard, footboard, safety sides and the mobilisation support. This also prevents limbs from reaching between the bars of the safety sides.



▲ The protective cover for the full-length safety sides can move up and down with the bars.



▲ The patient lifting pole helps the resident get comfortably positioned in the bed.



▲ The wall deflection roller protects the bed and the walls during horizontal and vertical movements.




▲ Fall protection mats made of 3 cm thick composite foam are recommended for maximum prevention of fall injuries.



▲ The wall spacer protects the bed, walls and furniture from collision damage.

ESSENTIALLY BETTER The spacer

- Wall spacers prevent damage due to collisions with furniture and power sockets when manoeuvring the bed

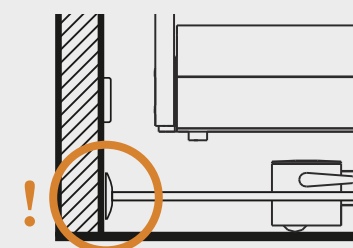

- Many different models to choose from
- Buffer absorbs the shock
- Optional discs distribute the pressure even more uniformly – ideal for lightweight wall constructions
- Dimensions to suit your bed

Technical data / Features

Model variant	Elvido vervo	Elvido movo
Dimensions and weights		
External dimensions (depending on the wooden surround)	approx. 100.3 x 211 cm	approx. 100.3 x 211 cm
Mattress base (mattress dimensions)	90 x 200 cm or 100 x 200 cm*	90 x 200 cm or 100 x 200 cm*
Mattress base division (BR/seat part/TR/LR)	approx. 85 / 15 / 40 / 60 cm	approx. 85 / 15 / 40 / 60 cm
Backrest length compensation	approx. 10 cm	approx. 10 cm
Safe working load	225 kg	225 kg
Ground clearance	approx. 15 cm	approx. 15 cm
Total weight	approx. 140 kg	approx. 140 kg
Drives		
Fully electric	✓	✓
Fully electric OpenBus technology	✓	✓
24-volt drive concept with external transformer	0	0
Electronic components		
LCD handset	0	0
Out-of-Bed system	0	0
Handset for tilting the head end down (Trendelenburg)	0	0
Safety side systems		
Integrated full-length safety sides	✓	✓
Integrated combined safety sides	0	0
Split safety side with Vario Safe	0	0
Side panel without safety side	0	0
Protective height of safety sides	41 cm	41 cm
Mattress bases		
Mattress base with optimal anthropometric dimensions (85/15/49/60 cm)	✓	✓
Hygiene mattress base	✓ polypropylene, removable	✓ polypropylene, removable
Comfort mattress base (Not with 100 cm mattress base)	0	0
Mattress base width 100 cm*	0	0
Wire mesh mattress base	-	-
4-section mattress base	✓ electrically adjustable	✓ electrically adjustable
Mattress base tiltable to reverse-Trendelenburg position	✓	✓
Mattress base tiltable to Trendelenburg position	0	0
Castors		
Castor	✓ double castor Ø 50 mm	✓ double castor Ø 100 mm
2x2 castor brakes	✓	-
4 central locking castors	0	✓
Wall deflection systems		
Wall deflection roller, horizontally effective	0	0
Wall deflection roller, horizontally and vertically effective	0	0
Wall spacer on chassis	0	0
Adjustment ranges		
Height adjustment	approx. 25 – 80 cm	approx. 28 – 83 cm
Backrest adjustment to 70°	✓	✓
Thigh rest adjustment to 40°	✓	✓
Reverse-Trendelenburg position	✓ up to 14°	✓ up to 14°
Trendelenburg position with additional handset	0 up to 14°	0 up to 14°
Miscellaneous		
Ground clearance for patient lift	✓	✓
Adapter sleeves at head end	✓	✓
Bed extension	✓ integrated	✓ integrated
Wood decors as shown in brochure	✓	✓
Wood decors as shown in brochure	✓	✓
Argentum metal finish	✓	✓
Auto contour setting	✓	✓
Side panel behind the safety sides	✓	✓
Accessory equipment		
Under bed light	0	0
Adapter sleeves at foot end	0	0
Integrated bed extension with telescopic safety side bars and side panels	0	0
Linen holder	0	0
Staining of wooden surround	0	0
Integrated light connection	0	0
Integrated battery	0	0

* Only with headboards: Alero, Pino, Lito, Lito reha and Podego

Elvido rega	Elvido forto
approx. 100.3 x 211 cm	approx. 110.3 x 211 cm
90 x 200 cm	100 x 200 cm*
approx. 71 / 15 / 41 / 61 cm	approx. 85 / 15 / 40 / 60 cm
-	approx. 10 cm
225 kg	260 kg
approx. 15 cm	approx. 15 cm
approx. 150 kg	approx. 150 kg
-	✓
-	✓
-	0
-	0
-	0
-	0
✓	✓
0	0
-	-
0	0
38 cm	41 cm
-	✓
-	✓ polypropylene, removable
-	-
-	✓
✓	-
✓ mechanically adjustable	✓ electrically adjustable
0	✓
0	0
✓ double castor Ø 50 mm	✓ Cover castor Ø 125 mm
-	-
✓	✓
0	0
0	0
0	0
approx. 39 – 80 cm	approx. 39 – 80 cm
✓	✓
✓	✓
0 up to 10°	✓ up to 14°
0 up to 14°	0 up to 14°
✓	✓
✓	✓
0 attachable with screws	✓ integrated
✓	✓ Choose from Lito, Lito reha, Pino, Alero or Podego
✓	✓
✓	✓
-	✓
-	✓
-	0
-	0
-	0
0	0
0	0
0	0
-	0
-	0

ESSENTIALLY BETTER

Elvido – the safety standards

- EN 14971
Risk assessment for medical devices
- EN 60601-1
Medical electrical equipment, general requirements for safety
- EN 60601-1-1
Electromagnetic compatibility
- DIN EN 60601-2-52
Medical electrical equipment



✓ = Standard feature
0 = Available as an option
- = Not available

Custodians of values Shapers of the future

STIEGELMEYERGROUP

Custodians of values

Our company was founded in Herford in 1900 and is still a family-run enterprise today. Respect and a sense of responsibility towards our customers, employees and partners form the basis for our day-to-day work. Based on these values and our many years of experience, we offer our customers all over the world high-quality products, solutions and services to meet the needs of the care and healthcare sectors.

Shapers of the future

In order to cope with rapid changes in the care and healthcare sectors, we develop all our products ourselves. Creating innovations and visions of what care will be like in future are the goals we pursue. We maintain regular contact with our customers in order to support them in their daily tasks. Implementing tomorrow's trends today is a successful tradition of ours.

Our expertise

Hospital

Nursing home

Homecare

Lifestyle beds

Furniture systems

Service



The Stiegmeyer-Group is your international partner for hospitals and care facilities and, with the Burmeier brand, also for caring in the home. We support our customers with practice-oriented products, solutions and services that we are constantly developing further.

- Founded in 1900 and still family managed
- Over 1,100 employees worldwide
- All core products manufactured in proprietary factories
- Product development and production in Germany
- Service, training and process optimisation at customer's premises
- International certification and export to more than 60 countries
- Distribution companies located in Belgium, Finland, France, Netherlands, Poland and South Africa

Our brands


STIEGELMEYER


BURMEIER



STIEGELMEYERGROUP

Hauptsitz / Headquarters

Deutschland / Germany
Stieglmeyer GmbH & Co. KG
Ackerstraße 42
32051 Herford
Phone +49 (0) 5221 185 - 0
Fax +49 (0) 5221 185 - 252
info@stieglmeyer.com
www.stieglmeyer.com

France
Stieglmeyer SAS
8 Allée des Chevreuils
69380 Lissieu
Phone +33 (0) 4 7238 - 5336
Fax +33 (0) 4 7238 - 5481
contact@stieglmeyer.fr
www.stieglmeyer.fr

South Africa
Stieglmeyer Africa (PTY) Ltd
The Estuaries · Century City
Cape Town, 7441
Phone +27 (0) 21 555 - 1415
Fax +27 (0) 21 555 - 0639
info.southafrica@stieglmeyer.com
www.stieglmeyer.co.za

Nederland / Netherlands
Stieglmeyer BV
Peppelenbos 7
6662 WB Elst
Phone +31 (0) 345 581 - 881
info@stieglmeyer.nl
www.stieglmeyer.nl

Suomi / Finland
Stieglmeyer Oy
Ahjokatu 4 A
15800 Lahti
Phone +358 (0) 3 7536 - 320
Fax +358 (0) 3 7536 - 330
stieglmeyer@stieglmeyer.fi
www.stieglmeyer.fi

Polska / Poland
Stieglmeyer Sp. z o.o.
86-212 Stolno
Grubno 63
Phone +48 56 6771 - 400
Fax +48 56 6771 - 410
info.stolno@stieglmeyer.pl
www.stieglmeyer.pl



Finden Sie Ihren Ansprechpartner auf unserer Website:
Find your local partner on our website:
www.stieglmeyer.com/partner

stieglmeyer.com