


Mobilo



Clinic | Care | Homecare | Assist

The
stretcher
for flexible use



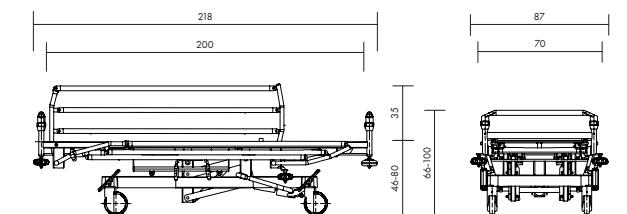

The Mobilo – more than meets the standard

The Mobilo stretcher serves many important purposes at once. As a transport stretcher for outpatient use, it is designed to bring patients swiftly and safely from one place to another. With its extensive features, it additionally meets standard DIN EN 60601-2-52 for beds. Therefore, it can be used as a daybed or couch in day hospitals, if required. This means less repositioning of patients and less pain. Depending on their diagnosis and state of health, patients are able to spend prolonged periods in the Mobilo without constant surveillance.

In addition, the Mobilo facilitates the work of nursing staff. Its central castor locking system can be operated with brake levers on all 4 castors. Especially in stressful situations, nursing staff can brake or move the stretcher from any direction.

As a stretcher and daybed, the Mobilo offers patients particularly high safety and comfort. This is ensured by features such as the 35 cm high safety sides on either side of the stretcher and the comfortable cold foam mattress.



1.



2.



3.




4.



1.



2.

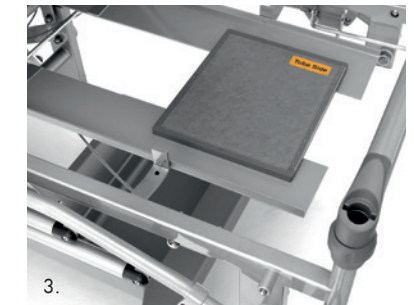

3.

Safety and comfort

The Mobilo **mattress base** is available in a 2-section or 3-section version. With its 70 x 200 cm, it is very generously sized for a stretcher. The mattress base height and position can be flexibly adjusted from both sides – from the treatment position right up to a comfortable sitting position (image 1).

The pivoting-down **safety sides** offer the patient a high level of safety (image 2). They present additional relief for hospital staff as depending on the situation the patient does not require constant surveillance.

The foot pedal raises the mattress base easily from **47 cm to 79 cm** (image 3). This allows patients to

get in and out of bed easily and provides an ergonomic working height for staff.

In an emergency, the Mobilo can be swiftly tilted to a **Trendelenburg position** thanks to its easy-to-operate controls (image 4).

The optional **resuscitation support** makes the stretcher even more versatile. It is arranged at the head end under the mattress base and can be swivelled downwards. During resuscitation, it provides a vibration-free, stable base.

Extras for many requirements

With a versatile range of accessories and numerous details, the stretcher can be adjusted to many demands of the daily hospital routine.

The wide, ergonomically designed **push grips** at the head and foot end of the stretcher make it easy to manoeuvre swiftly and easily (see large illustration).

The **paper roll holder** can be fitted either at the head or foot end and ensures particularly hygienic use of the mattress (image 1).

At the head and foot end, the stretcher is equipped with **sliding rails** to allow accessories such as pumps, stands or the oxygen supply system to be attached (image 2).

Optionally, the Mobilo can be equipped with a **holder for X-ray cassettes** (image 3). As the backrest can be x-rayed, there is no need for time-consuming and stressful repositioning of the patient.



Care in every situation

All care items required for treatment can be securely attached to the Mobilo both quickly and easily. This ensures that the Mobilo can be used flexibly as a stretcher or day couch.

The easy-to-attach **tray** facilitates patient care without the need for repositioning (see large illustration).

The optionally available **infusion stands** are of double telescopic design and can be fitted to all four corners of the stretcher (image 1).

A storage tray underneath the Mobilo offers an easily accessible space for an **oxygen bottle including a belt** and for the patient's personal items (image 2).

Essential care accessories can be versatilely attached to the **supply rails** on both sides of the stretcher (image 3). Further accessories are available on request.

Technical data

	Mobilo	
Dimensions and weights		
External dimensions	87 x 218.5 cm	
Mattress base (mattress dimensions)	70 x 200 cm	
Mattress height	10 cm	
Safe working load	200 kg	
Maximum lifting load	250 kg	
Ground clearance with mattress base in lowest position	approx. 9 cm	
Total weight	approx. 130 kg	
Mattress base in 2 sections	74 cm-117.5 cm	
Mattress base in 3 sections	74 cm – 13 cm – 37.7cm – 61.8 cm	
Protection height of safety sides	35 cm	
Adjustment options		
Tilting to Trendelenburg and reverse-Trendelenburg position	12°	
Mattress base height	approx. 47 – 79 cm	
Backrest angle	70°	
Thigh-rest angle (with mattress base in 4 sections)	45°	
Equipment and options		
Safety sides	with safety sides on both sides	●
Basic colour	Crystal white NCS S0500N	●
	Argentum similar to RAL 9006	○
Mattress bases	2-section mattress base	●
	3-section mattress base	○
Wall deflection rollers	6, horizontally acting	●
	6, of which 2 horizontally and vertically acting	○
	6, horizontally and vertically acting	○
Castors with central braking system	Double castor Ø 150 mm	●
	Integral castor Ø 200 mm	○
Track castor positions	Head end, right	●
	Head end, left	○
	Foot end, right	○
	Foot end, left	○
Conductive castor coating	1 castor	●
	2 castors	○
	4 castors	○
Further equipment	Food and deposit tray	○
	Holder for X-ray cassettes under the mattress base	○
	Resuscitation support	○

● = standard ○ = optional

Mattress base

- 2-section or 3-section all-plastic mattress base (HPL).

Fastening options for supply accessories

- Two locating sleeves for infusion stands at head and foot end, respectively, integrated in the handles
- Supply rail on both sides with universal holders underneath the mattress base

Head and footboards

- Handle removable with screw connection
- Suitable to accommodate infusion stands

Materials

- Chassis and mattress frame: Welded steel construction with durable polyester powder coating
- Mattress base parts: All-plastic (HPL)
- Handles: Polyamide 6, POM, stainless steel

Protection against damage

- Edge protection integrated into the safety sides
- 6 wall deflection rollers, also acting horizontally and vertically

Ihr Bett – unsere Leidenschaft
Your bed – our passion

Finland / Suomi

Stieglmeyer Oy
Ahjokatu 4 B
15800 Lahti
Phone +358 (0) 3 7536 - 320
Fax +358 (0) 3 7536 - 330
E-Mail stieglmeyer@stieglmeyer.fi
finland.stieglmeyer.com

Netherlands / Nederland

Stieglmeyer BV
Peppelenbos 7
6662 WB Elst
Phone +31 (0) 345 581 - 881
E-Mail info@stieglmeyer.nl
netherlands.stieglmeyer.com

Poland / Polska

Stieglmeyer Sp. z o.o.
86-212 Stolno, Grubno 63
Phone +48 56 6771 - 400
Fax +48 56 6771 - 410
E-Mail info.stolno@stieglmeyer.pl
poland.stieglmeyer.com

France

Stieglmeyer SAS
451 Chemin de Champivost
69 760 Limonest
Phone +33 (0) 4 7238 - 5336
Fax +33 (0) 4 7238 - 5481
E-Mail contact@stieglmeyer.fr
france.stieglmeyer.com

South Africa

Stieglmeyer Africa (PTY) Ltd
The Estuaries, Century City, Cape Town, 7441
Phone +27 (0) 21 555 - 1415
Fax +27 (0) 21 555 - 0639
E-Mail info.southafrica@stieglmeyer.com
south-africa.stieglmeyer.com

Technische Änderungen vorbehalten. Leichte Farbabweichungen der Originale sind drucktechnisch bedingt. Stand 12/2016. Version 3. Artikel-Nr. 600506.
Subject to technical changes without notice. The shown product colours may differ slightly from the actual product. Last updated 12/2016. Version 3. Item no. 600506.

Finden Sie Ihren Händler auf unserer Webseite:
Find your local partner at our website:
stieglmeyer.com/partner

Stieglmeyer GmbH & Co. KG

Ackerstraße 42, 32051 Herford, Germany
Phone +49 (0) 5221 185 - 0
Fax +49 (0) 5221 185 - 252
E-Mail info@stieglmeyer.com
www.stieglmeyer.com

